

## **SYRINGE ACCU-GEL SYSTEM1 & SYSTEM2**

**Mix the System1 Syringe Accu-Gel** using 13cc's of room temperature (70° F) water (*bottom line on the left side of the measuring vial*).

**Mix the System2 Syringe Accu-Gel** using 27cc's of warm (80° F) water (*bottom line on right side of the measuring vial*).

The Syringe Gel is applied to the buccal and labial vestibules with a catheter tip syringe. In SYSTEM2 the Syringe Gel is applied to the teeth with the Pre-Sure Tip. It is the Syringe Gel that eliminates air bubbles and produces the delicate bordermolding in the maxillary impression. The nice thing about these materials is that they are easily moldable, yet it doesn't tend to flow of its own weight (thixotropic).

### **Tray Accu-Gel**

**Mix the System1 Tray Accu-Gel** using 41cc's of room temperature (70° F) water (*top line on the left side of the measuring vial*).

**Mix the System2 Tray Accu-Gel** using 52cc's of warm (80° F) water (*fill vial level with top*).

The Tray Accu-Gel is loaded into the tray and brought to the mouth after the placement of the Syringe Gel. Loaded Accu-Trays are always gently placed into the mouth (*not seated*) from the front to the back. This gentle placement of the tray supports, but does not displace, the Syringe Gel in the vestibules. This gentle placement of the tray also minimizes displacement of tissues overlaying the alveolar ridge.

## **LOWER SYSTEM1 & SYSTEM2 ACCU-TRAYS**

**Take a close look at the enclosed Accu-Trays.**

**SYSTEM1-** The high distal rise enables you to capture the entire retromolar pad, and the long lingual flanges push the sublingual gland and mylohyoid muscle out of the way for a good clean impression of the lingual surface of the mandible in the first impression.

**SYSTEM2-** The long lingual flanges push the sublingual gland and mylohyoid muscle out of the way for a good clean impression of the lingual surface of the mandible *without* having to push the posterior of the tray downward. This automatically keeps the retromolar pads up inside your impression. These impressions are so complete and so accurate, that over 90% of Accu-Dent customers use their first set of impressions as final impressions.

By the time you've done all of the above, you will have a good understanding of the concept behind Accu-Dent impressions. System2 is a little different than System1, but the concept and the high quality of the results are identical.

**In addition, all Accu-dent products are backed by our:**  
**60-Day "Try-It-In-Your-Own-Operator" Unconditional Satisfaction,**  
**Money-Back Guarantee.**

If you would like to try one or both of these exciting products, please call **Ragle Dental Laboratory** or e-mail us at: **info@raglelab.com**